

RapidResults Quick Reference Guide v6.0

Logging onto RapidResults

1. Open your internet browser, and enter the PowerReader URL.
2. Enter your PowerReader credentials and click **Log In**.
3. Click the RapidResults tile to launch the program.

Opening a Study














When RapidResults loads, you will be taken to the **My Latest Exams** tab. This tab contains a list of your most recent exams. To open a study from this list, you simply select it. If the study you wish to open is not displayed on this list you may perform a search to locate it.


Searching for Studies

1. (Mouse) Click the **Search** tab. / (Touch) Tap **Menu**, then tap **Search**.
2. Enter search criteria in the provided fields (i.e. Last Name, Patient ID, etc.)
3. Click/Tab **Search**.

NOTE: RapidResults will automatically detect if you are on a touchscreen enabled device, such as a tablet or smartphone. If your device supports both touch and mouse input, you may switch between these modes by clicking the **Touch Mode** or **Mouse Mode** buttons at the top-right of your screen.

Image Viewing Tools



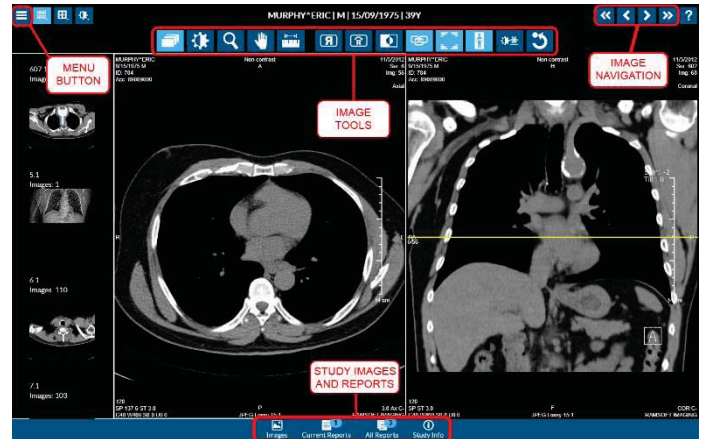

-  **Scroll**
Drag up and down to scroll through images.
-  **Window/Level Tool**
Drag up and down to adjust the level and left and right to adjust the Window value.
-  **Zoom Tool**
Drag up and down to zoom in and out.
-  **Pan Tool**
Drag to reposition the image within the viewport.
-  **Measure Distance**
Drag from one point to another to measure the distance.
-  **Flip Tool**
Flips the selected image horizontally.
-  **Rotate Tool**
Rotates the selected image 90° clockwise.
-  **Invert Tool**
Inverts the contrast of the selected image.
-  **Link Tool**
Toggles plane linking on and off.
-  **Toggle Overlays**
Toggle the Patient, Image and User Annotations on or off.
-  **Show/Hide Localizer Lines**
Toggles the localizer (cut) line on or off.
-  **Clamp Mode**
Toggles Window/Level clamping on or off.
-  **Reset Image**
Resets an image to its original state.



RapidResults Image Viewer in Touchscreen Mode

Image Navigation

The images for a study are grouped into framesets and can be found on the left-hand sidebar of the image viewer. To display a frameset, simply select a thumbnail. Optionally, use the buttons below to navigate through them.

- | | | | |
|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|
|  |  |  |  |
| Previous Frameset | Previous Frame | Next Frame | Next Frameset |
| Hangs previous frameset in the active viewport. | Moves active frameset backward one frame. | Moves active frameset forward one frame. | Hangs next frameset in the active viewport. |

Viewing and Printing Reports

To view and print reports you will need to have a PDF viewer installed on your system. You should consult your specific product's documentation for details on printing. These instructions apply to users of Adobe Acrobat™.

1. Open a study.
2. Click/Tab **Current Reports** or **All Reports** button.
3. Select a report from the list. The report will be displayed.
4. Right-click the report and select **Print** from the pop-up menu.
5. Select your printer and click/tap **Print**.

Closing a Study

1. (Mouse) Close the active browser tab.
2. (Touch) Tap **Menu**, then tap any link (i.e. My Latest Exams).

Requesting an Appointment

If the facility supports scheduling you may request new appointments for your patients via the **Request an Appointment** screen. These are only requests and will need to be confirmed by the imaging facility's scheduling staff.

1. (Mouse) Click the **Request an Appointment** tab. / (Touch) Tap **Menu**, then tap **Request an Appointment**.
2. Enter a **Last Name**, **First Name**, and **Phone #**.
3. Select the **Birth Date** and **Imaging Facility** field.
4. Click/Tab **Add Procedure**, then select a **Modality** and **Procedure Code**.
NOTE: The list of available modalities will be Facility dependent.
5. Repeat Step #3 for each procedure requested.
6. Enter a **Reason For Study** and **Referring Physician**.
7. (Optional) Enter an **Auth. #** and select a **Requested Appt. Time** and **Date**.
8. Click/Tab **Submit**.